

응용 자료

ACQUITY UPLC I-Class/Xevo TQD IVD System: Analytical Performance for Fat Soluble Vitamins

Waters Corporation

본 응용 개요서는 구체적인 실험 내용을 포함하지 않습니다.

체외 진단용입니다. 일부 국가에서는 사용할 수 없습니다.

Introduction

The Waters ACQUITY UPLC I-Class/Xevo TQD IVD System enables the quantification of organic compounds in human biological liquid matrices.

This document describes a test of the analytical performance of the ACQUITY UPLC I-Class/Xevo TQD IVD System for the analysis of vitamin A and vitamin E in serum.



The Waters ACQUITY UPLC I-Class/Xevo TQD IVD System.

Experimental

The ACQUITY UPLC I-Class/Xevo TQD IVD System was controlled by MassLynx IVD Software (v4.2) and the data was processed using the TargetLynx Application Manager. In-house calibrators and quality controls were prepared by spiking commercially available reference material in serum. The samples were processed using the following conditions:

Sample Preparation Conditions

A 100- μ L sample was processed with ethanol and water and centrifuged. The supernatants were diluted then loaded onto Oasis PRiME HLB μ Elution plates, washed, and eluted prior to analysis.

LC Conditions

| | |
|-----------------|---|
| Column: | ACQUITY UPLC HSS PFP, 1.8 μ m, 2.1 mm \times 50 mm |
| Mobile phase A: | 2 mM ammonium acetate + 0.1% formic acid in water |
| Mobile phase B: | 2 mM ammonium acetate + 0.1% formic acid in methanol |
| Flow rate: | 0.4 mL/min |
| Gradient: | 65–98% B over 2 minutes, 98% B for 0.55 minutes, 65% A for 0.95 minutes |

MS Conditions

| | |
|-------------------|----------------------------------|
| Resolution: | MS1 (0.75 FWHM), MS2 (0.75 FWHM) |
| Acquisition mode: | MRM |
| Polarity: | ESI+ |

Results and Discussion

Chromatograms illustrating the chromatographic selectivity of vitamins A and E are shown in Figures 1 and 2. Performance characteristics of vitamins A and E on the ACQUITY UPLC I-Class/Xevo TQD IVD System are shown in Table 1.

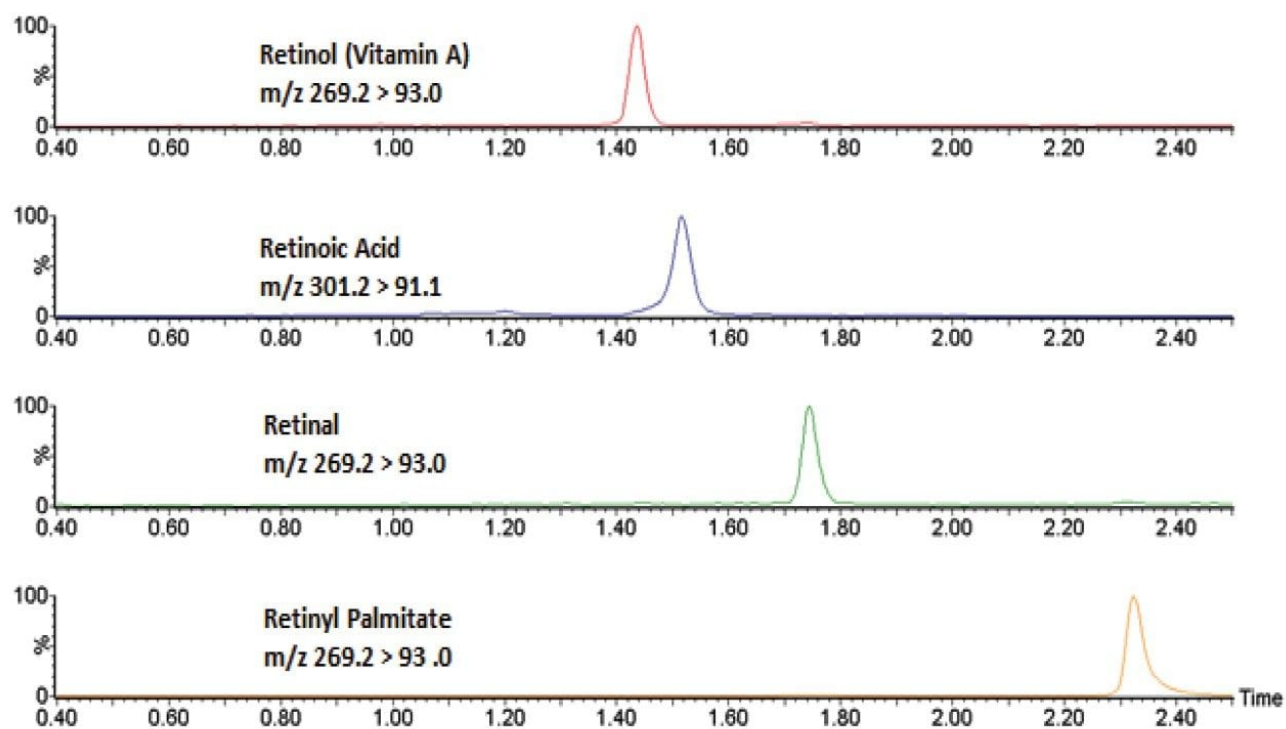


Figure 1. Chromatographic separation of vitamin A from metabolites and structurally similar compounds using the ACQUITY UPLC I-Class/Xevo TQD IVD System.

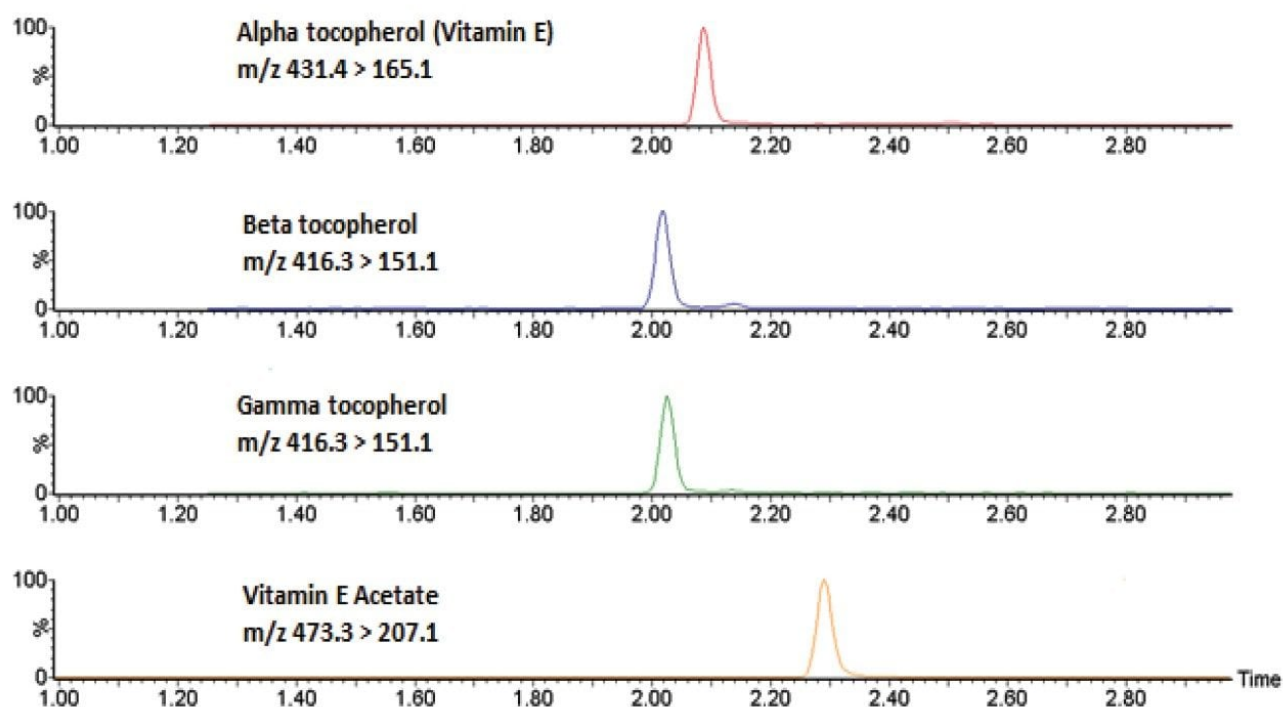


Figure 2. Chromatographic separation of vitamin E from structurally similar compounds using the ACQUITY UPLC I-Class/Xevo TQD IVD System.

| Compound | Range (ng/mL) | LLOQ (ng/mL) | %RSD at LLOQ | Total precision | Repeatability | EQA mean bias |
|-----------|---------------|--------------|--------------|-----------------|---------------|---------------|
| Vitamin A | 100–2000 | 50 | 9.6% | ≤5.4% | ≤4.8% | -7.0% |
| Vitamin E | 2100–21100 | 1100 | 14.7% | ≤6.9% | ≤3.9% | -10.9% |

Table 1. Performance characteristics of vitamins A and E. Range defined by linear fit where $r^2 > 0.99$. LLOQ defined by $S/N (PtP) > 10$ and $\%RSD \leq 20\%$; $\%RSD$ at LLOQ determined through analytical sensitivity experiments performed over three occasions ($n=30$). Total precision and repeatability of QCs performed over five occasions in serum ($n=25$). EQA mean bias (Bland-Altman agreement) determined by comparison of obtained values to all laboratory trimmed mean.

Conclusion

The Waters ACQUITY UPLC I-Class/Xevo TQD IVD System has demonstrated the capability to deliver analytically sensitive, accurate, and precise performance for vitamins A and E in serum.

Disclaimer

The analytical performance data presented here is for illustrative purposes only. Waters does not recommend or suggest analysis of the analytes described herein. These data are intended solely to demonstrate the performance capabilities of the system for analytes representative of those commonly analyzed using liquid chromatography and tandem mass spectrometry. Performance in an individual laboratory may differ due to a number of factors, including laboratory methods, materials used, intra-operator technique, and system conditions. This document does not constitute a warranty of merchantability or fitness for any particular purpose, express or implied, including for the testing of the analytes in this analysis.

Featured Products

[MassTrak ACQUITY UPLC I-Class PLUS/Xevo TQD IVD System </nextgen/kr/ko/products/in-vitro-diagnostics-solutions/lc-ms-ms-ivd-systems-portfolio/masstrak-acquity-uplc-i-class-plus-xevo-tqd-ivd-system.html>](https://www.waters.com/waters/nav.htm?cid=134834177)

[MassLynx \(IVD\) Mass Spectrometry Software with TargetLynx \(IVD\) <https://www.waters.com/waters/nav.htm?cid=134834177>](https://www.waters.com/waters/nav.htm?cid=134834177)

720006728, December 2019



© 2025 Waters Corporation. All Rights Reserved.

[이용 약관](#) [개인정보처리방침](#) [상표](#) [채용정보](#) [법적 고지](#) [및 개인정보 보호 고지](#) [쿠키](#) [쿠키 기본설정](#)